

## No Sting Barrier Film Swab and Wipe

### Instructions for Use

#### 1.0 Product Description

**Halyard** Barrier Film is a polymeric solution which forms a uniform film when applied to the skin using a swab (foam applicator) or wipe. The product is dispersed in a non-stinging solvent, which dries rapidly. The film is colourless, transparent, and possesses good oxygen and moisture vapour permeability. **Halyard** Barrier Film helps to protect intact or damaged skin from irritation caused by urine and/or fecal incontinence, digestive juices, wound drainage.

#### 2.0 INTENDED PURPOSE

**Halyard** Barrier Film is a liquid intended for use as a film-forming product that upon application to intact or damaged skin forms a long lasting waterproof barrier, which acts as a protective interface between the skin and bodily wastes, fluids. It is intended as a primary barrier against irritation from body fluids and can be used on adults and children.

##### 2.1 INTENDED POPULATION

The product may be used on adults, children and infants over 12 weeks old.

##### 2.2 INTENDED USERS

Intended for use by health professionals and may be used in a hospital, community and home setting.

#### 3.0 INDICATIONS

The products are indicated for protecting intact or damaged skin from body fluids in the areas of:

- Incontinence skin care
- Peri-stomal skin protection
- Peri-wound skin protection

#### 4.0 CONTRAINDICATIONS

**Halyard** Barrier Film is not to be used as the only covering in situations that require dressing protection from bacterial contamination / penetration e.g. intravenous therapy catheter sites and full or partial thickness wounds. **Halyard** Barrier Film is not to be used on infected skin.

#### 5.0 WARNING



**Halyard** Barrier Film, in liquid form, is flammable; use in a well ventilated area. Avoid using around flames and sources of ignition. Keep out of reach of children.



Do not use if pack is damaged or open.

## 6.0 INFORMATION FOR USE

- Skin should be clean and dry prior to application of Halyard Barrier Film.
- Remove foam applicator from sachet and apply a uniform coating of film over entire area of concern.
- Visually inspect Sachet for damage prior to opening
- Follow local protocols on the handling of sterile product.
- If an area is missed, reapply to that area only after first application of Halyard Barrier Film has dried (approximately 30 seconds).
- If Halyard Barrier Film is applied to area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow the film to thoroughly dry before returning to normal position.
- When used under adhesive tapes, dressings, or devices:
  - (a) Allow Halyard Barrier Film to thoroughly dry before covering with dressing or adhesive products.
  - (b) Reapplication of Halyard Barrier Film is necessary each time dressing and/or adhesive products are changed; the barrier film is removed by the adhesive.
- When used as a protectant against body fluids, feces, or urine etc., and no adhesive products are applied to skin:
  - (a) Reapplication of Halyard Barrier Film is recommended every 24-72 hours, depending on frequency of cleaning.
  - (b) In extreme cases (e.g. constant diarrheal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
- If desired, the film can be removed by using most medical adhesive removers as directed. Clean and dry the involved area and reapply Halyard Barrier Film.

### Ingredients

- Disiloxane, Acrylate Copolymer

## 7.0 CLINICAL BENEFITS /SPECIAL NOTES

Barrier films offer protection to the skin against subsequent moisture contact. The Barrier film forms a thin layer, repelling moisture and potential irritants, thus protecting the skin.

### Caution

1. Should redness or any other signs of irritation appear, or if redness or signs of irritation persist, cease the use of Halyard Barrier Film and consult a physician.
2. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of Halyard Barrier Film.

## 8.0 STORAGE







Store away from direct sunlight at ambient temperature and humidity. Store in a dry place.

**9.0 DISPOSAL**

This must be disposed of in compliance with anti-pollution and other laws of the country concerned. Contact your local service providers for further information

The **Halyard** Barrier Film Range can be disposed of based on the hospital or healthcare professional advice.

**10.0 SYMBOLS ON LABELLING**

WARNINGS	
	Flammable
	Do not use if the pouch is damaged or opened.
STORAGE	
	
STERILE	
  	

**11.0 PRESENTATION**

**Halyard** Barrier Film is available in the following formats:

Cat. No.	Size	No. /Box
MEDLO50015	0.75ml No Sting Barrier Film Wipe	25
MEDLO50025	1ml No Sting Barrier Film Wipe	25
MEDLO50125	1ml No Sting Barrier Film Swab	5
MEDLO50225	3ml No Sting Barrier Film Swab	5

**12. COMPLAINTS**

In USA, please call 1-844-425-9273 • [halyardhealth.com](http://halyardhealth.com)



[halyardhealth.com/information](http://halyardhealth.com/information)

Manufactured for Owens & Minor Distribution, Inc., 9120 Lockwood Boulevard, Mechanicsville, VA 23116, USA

\*Registered Trademark or Trademark of O&M Halyard, Inc., or its affiliates. © 2021. All rights reserved.

Made in Ireland