

Askina Barrier Cream

en Moisture Barrier Cream
de Feuchtigkeitspendende Hautschutz-Creme
cs Crema de barrera hidratante
fr Crème protectrice hydratante
it Crema protettiva idratante
nl Hydraterende barriercrème
ar Zvhčujúci bariérny krém
cz Fugtgighedsbevarende barriercreme
el Κρέμα Φρουγού Υγρασίας
et Niiskuskaitse kreem
fa زیست گرداننده اندامی خالق چشم
fi Kosteussuojaivoide
he 保濕隔離乳
kr 保湿度アリーラー_L
km Krin Penghalang Pelembab
no Fuktighetsgivende barrierkrem
sv Navilisajacy krem ochranny
uk Крем-протектор для тіла
ro Cremă de tip barierică împotriva umidității
sk Zvhčujúci ochranný krém
tr Krem za zaštitu od tečnosti
vn Krem na ochranu vody
tr Nemlendirici Bariyer Kremi
uk Зволюючий захисний крем
km Kem chđn hám loét
zh 保湿阻隔霜

Avery Dennison Medical Ltd.
IDA Business Park,
Ballinalacke Road,
Longford, N39 DX73,
Ireland

B|BRAUN

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION
Barrier Cream is a water concentrated cream that provides protection from skin by body fluid. It protects dry, chafed, red, or irritated skin by creating a protective barrier. It is a single use, lasting barrier.

Using this cream allows adhesives/adhesive strips to stick to the skin. Barrier Cream is a multiple use device that is intended for single patient use.

2. INTENDED PURPOSE

Barrier Cream acts as a mechanical barrier, and irritants associated with incontinence.

3. INTENDED POPULATION

The product may be used on adults, children and infants over 12 weeks old.

2.2 INTENDED USER

Intended for use by health professionals and should be used in a hospital, community and home setting.

3. INDICATIONS

Barrier Cream protects at-risk skin areas from damage and irritation associated with incontinence. The Barrier Cream preserves skin from damage associated with chronic incontinence. It shields the skin from damage and irritation associated with incontinence.

4. CONTRAINDICATIONS / SAFETY INFORMATION

Barrier Cream should not be used on infected skin. Barrier Cream should not be used where there is known allergic sensitivity to the ingredients.

5. WARNINGS

Barrier Cream may interfere with adherence of adhesive products. If the condition worsens or does not improve within 7 days, consult a physician. Keep out of reach of children under 12 years old. Avoid contact with eyes.

6. INFORMATION FOR USE

Gently cleanse skin area before. Spread Barrier Cream to cover entire affected area, as necessary. If the feel is oily, too much has been applied.

7. CLINICAL BENEFITS / SPECIAL NOTES

The Barrier Cream forms a barrier that acts as a physical shield between the skin and bodily wastes and fluids. The product is designed to assist barrier resistance against irritation from bodily wastes.

8. STORAGE

Store dressings away from direct sunlight at ambient temperature and humidity. Product should be discarded 28 days after opening.

9. DISPOSAL

Product must be disposed of in compliance with local waste management regulations. Outer packaging may be recycled. Contact your local service providers for further information.

10. PRESENTATION

10g tube, 92g tube 1 Stück

11. CONFORMITY

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority concerned. Contact your local service providers for further information.

12. DISTRIBUTOR

B. Braun Hospicare Ltd.,
Collooney, Co. Sligo,
Ireland
Phone: +353 71 911 5064
Fax: +353 71 911 5064
www.bbraun.com/living-with-a-stoma
Made in Ireland



A1329-01 10/21
LFD-PKG-00072 Rev A

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

5038

