REVISION RECORD

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Ingredienser:
• Disiloxan, acrylatcopolymer

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Foam Applicator

Product Description

Sorbaderm Barrier Film is a polymeric solution which forms a uniform film when applied to the skin. The product is dispersed in a non-stinging solvent, which dries rapidly. The film is colourless, transparent, and possesses good oxygen and moisture vapour permeability. Sorbaderm Barrier Film helps to protect intact or damaged skin from irritation caused by urine and/or fecal incontinence, digestive juices, and wound drainage.

Indications

Sorbaderm Barrier Film is a liquid intended for external use as a film-forming product that, upon application to intact or damaged skin, forms a long lasting waterproof barrier which acts as a protective interface between the skin and bodily wastes.

It is intended to serve as a primary barrier against irritation from body fluids.

Contraindications:

Sorbaderm Barrier Film is not to be used as the only covering in situations that require dressing protection from bacterial contamination / penetration / e.g. intravenous therapy catheter sites and full or partial thickness wounds. Sorbaderm barrier film is not to be used on infected skin.

Warnings:

Sorbaderm Barrier Film, in liquid form, is flammable; use in a well ventilated area. Avoid using around flames and sources of ignition. Product is sterile unless open or damaged. Sorbaderm Barrier Film is for external use only.

Precautions: 1. Should redness or any other signs of irritation appear, or if redness or signs of irritation persist, discontinue use of Sorbaderm Barrier Film and consult a medical professional.

2. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of Sorbaderm Barrier Film.

3. 1ml and 3ml foam applicators are individually packaged for single use only. Do not use if the packaging is damaged or opened.

Directions for use:

- Skin should be clean and dry prior to application of Sorbaderm Barrier Film.
- Tear open pouch where indicated and remove the foam applicator, which has been pre-loaded with the barrier film liquid. Using the applicator, with the foam end in contact with the skin, gently apply a uniform coating of the liquid barrier film to the area of concern. If an area is missed, reapply to that area only after first application of Sorbaderm Barrier Rim has dried (approximately 30 seconds).
- If Sorbaderm Barrier Rim is applied to an area with skin folds or other skin-to-skin contact, make sure that all skin-to-skin areas are separated to allow the film to thoroughly dry before returning to normal position.
- When used under adhesive tapes, dressings, or devices:
 - (a) Allow Sorbaderm Barrier Film to thoroughly dry before covering with dressing or adhesive products.
 - (b) Reapplication of Sorbaderm Barrier Film is necessary each time dressing and/or adhesive products are changed; the barrier film is removed by the adhesive.
- When used as a protectant against body fluids, faeces, or urine etc., and no adhesive products are applied:
 - (a) Reapplication of Sorbaderm Barrier Film is recommended every 24-72 hours, depending on frequency of cleaning.
 - (b) In extreme cases (e.g. constant diarrhoeal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
- If desired, the film can be removed by using most medical adhesive removers as directed. Clean and dry the involved area and reapply Sorbaderm Barrier Film.

Ingredients:

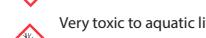
• Disiloxane, Acrylate Copolymer


Highly flammable liquid and vapor

Storage:

Store away from direct sunlight at ambient temperature and humidity.

Very toxic to aquatic life

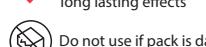Toxic to aquatic life with long lasting effects

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