REVISION RECORD

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Sorbaderm Barrier Cream IFU (Front)

en Sorbaderm® Barrier Cream

Product Description
Sorbaderm® Barrier Cream is a white concentrated cream that provides the skin with protection from body fluids. It also protects dry, chafed, red, or irritated skin by moisturising the skin and providing a long lasting barrier. Use of this cream still allows adhesive products to stick to the skin.

Indications:
Sorbaderm Barrier Cream is for the protection of intact skin areas at risk from damage and irritation. Sorbaderm Barrier Cream maintains and preserves intact skin from damage associated with chronic incontinence. Sorbaderm Barrier Cream may also be used to moisturise and protect severely dry skin. The cream will not reduce adhesion of medical tapes or adhesive dressings that may be placed over treated areas.

Warnings:
Sorbaderm Barrier Cream may increase the adherence of some adhesive products. If skin condition worsens or does not improve within 7 days, consult a physician. Keep out of reach of children. In case of ingestion, seek medical help. For external use only. Avoid contact with eyes.

Contra-indications
Sorbaderm Barrier Cream should not be used on infected skin. Sorbaderm Barrier Cream should not be used where there is known allergic sensitivity to the ingredients.

Directions for use:
• Gently cleanse skin area as required.
• Using Sorbaderm Barrier Cream sparingly, spread a thin layer over the entire affected area. If after application the feel is oily, too much cream has been applied.
• Repeat applications as required (or as directed by a healthcare professional).

Ingredients:
Purified Water, Ethylhexyl Isononanoate, Disiloxane, Polyisobutylene, VP Eicosene Copolymer, Hydroxyethyl Acrylate / Sodium Acryloyl Dimethyl Taurate Copolymer, Butylene Glycol, Oleth-3 Phosphate, Phenoxethanol, Hydrogenerated Styrene/Isoprene Copolymer, Allantoin, Carbomer, Triethanolamine, Methylparaben, Propylparaben, Disodium EDTA.

Storage:
Store away from direct sunlight at ambient temperature and humidity. Do not freeze.

Presentation	Cat. No.	Size	No./Box	Do not use if pack is damaged.
3026	Barrier Cream 2g Sachet		20	Medical Device Unique Device Identifier
3027	Barrier Cream 28g Tube		1	
3028	Barrier Cream 92g Tube		1	

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Avery Dennison Medical Ltd.
 IDA Business Park, Ballinalee Road,
 Longford, N39 DX73, Ireland
 LFD-PKG-000818-A

Distributed by:
 T.J. Smith and Nephew, Limited
 101 Hessel Road, Hull HU3 2BN
 England.
 Ordering Enquiries: +44 (0)800 015 7573

Sorbaderm Barrier Cream IFU (Back)

Blue = die line (does not print).

Sorbaderm IFU Graphics File Name			
N/A			
Raw Material #			
Full Scale*	1 Proof #		
N/A			
Bell-Mark Die #	N/A	3026, 3027, 3028	
		Tool Color Used	Finished Good Number(s)
		Artwork ID# LFD-PKG-000818-A	
PMS Color(s) Used		Printed by Avery Dennison Medical Ltd.	
		Notes	

*Scale as designed. Check here if printed at less than full scale.

RELEASE