

# Instructions for use

## ahc-Sensitive

### 1. PRODUCT DESCRIPTION

ahc-Sensitive is a perforated, gently adhering, non-irritating wound contact layer dressing, consisting of a fleece matrix that is coated on both sides with a soft silicone layer. ahc-Sensitive covers the wound and is not absorbent.

ahc-Sensitive serves as a protective barrier so that granulation tissue can form and grow. Wound rest is guaranteed and not disturbed, for example, by changing the absorbent secondary dressing too frequently. ahc-Sensitive lets exudate through and prevents maceration of the wound without impairing the function of the absorbent dressing. The wound environment is protected.

The advantages and functions of ahc-Sensitive are:

- Exudate can be drained through the perforated openings to an absorbent dressing
- pleasant wearing comfort due to an excellent adaptation to the natural contours of the body
- latex-free and non-irritating
- minimizes the trauma when changing the dressing and reduces the frequency of the required dressing change (wound rest is guaranteed)
- easy application and removal

### 2. INDICATIONS

ahc-Sensitive is a non-invasive wound dressing that can be used in long-term therapy for patients with damaged dermis and secondary wound healing.

#### 2.1 Target group

##### 2.1 Target group

Patients with a damaged dermis who have to heal secondarily are addressed. ahc-Sensitive is intended for use in a wide variety of wounds including skin grafts, donor sites, post-operative wounds, skin cracks, cuts and abrasions, stage I-IV pressure sores, venous and arterial ulcers, second degree burns.

#### 2.2 User target group

Medical professionals are intended for the application.

### 3. INFORMATION

ahc-Sensitive is intended for use in a wide variety of wounds including skin grafts, donor sites, post-operative wounds, skin cracks, cuts and abrasions, stage I-IV pressure sores, venous and arterial ulcers, second degree burns.

The sales company is Akanni Healthcare GmbH Iphofen.

### 4. CONTRAINDICATIONS / SAFETY INFORMATION

ahc-Sensitive is contraindicated:

- If signs of infection appear or the wound worsens unexpectedly, stop using the product and consult a doctor.
- Do not use ahc-Sensitive on infected wounds or third degree burns.
- ahc-Sensitive should not be used for known allergies to silicone or any of the ingredients.
- ahc-Sensitive should not be changed 5 days after the fixation of the skin graft.

### 5. WARNINGS

General safety information for medical devices:



ahc-Sensitive is a sterile product for single use. The reuse of disposable items carries a potential risk for the patient or user. This can lead to contamination and / or impaired functionality. Contamination and / or limited functionality of the association can lead to injury, illness or death of the of the patient.



A damaged or opened package is considered to be non-sterile.

## 6. PACKAGE LEAFLET

Check the packaging for any damage before you are opening.

### 6.1 Wound preparation

a) Before use and when changing the dressing, wound cleaning according to the medical standard and / or wound debridement are recommended to remove exudate, fibrinoid deposits and other contaminants. Dry the wound area.

### 6.2 Application

a) Take the wound dressing out of the peel bag. ahc-sensitive can be cut under sterile conditions if necessary before the outer films are removed. An overlap of the wound of at least 2 cm is recommended.

b) Remove the first film and place ahc-sensitive over the entire wound area.

c) Pull back the second slide while holding the dressing in place.

Both sides of ahc-Sensitive can face the wound. ahc-Sensitive should not be used twice, as this can lead to occlusion.

d) If necessary (depending on the exudation) put on a secondary absorbent dressing and / or fixation.

### 6.3 Dressing change

Depending on the wound situation, ahc-Sensitive can be worn for up to 7 days. Note the consistency of the exudate so that it can be drained freely through the pores away from the wound.

Remove the secondary dressing as needed and leave ahc-Sensitive on the wound to ensure wound rest.

To remove ahc-Sensitive, simply gently peel the matrix off the wound and skin. If necessary, the bandage can be moistened before.

## 7. CLINICAL ADVANTAGE / SPECIAL CONDITIONS

ahc-Sensitive is considered a safe and effective dressing for the treatment of numerous wounds. The many advantages are less pain when changing bandages, less adhesion and the ability to adapt to the body shapes, which results in improved healing of the wound and wound environment without damage, or even peri-wound.

## 8. STORAGE







Protect the bandages from direct sunlight and moisture / humidity, and store the bandages at room temperature.

ATTENTION: Do not use if the packaging or the peel bag is damaged. Use the products only up to the expiry dates stated on the packaging.

## 9. SYMBOLS FOR LABELING

WARNINGS	
	Do not use if the package is damaged.



	Medical device
	Sterile barrier system / sterile packaging
<b>STERILIZATION</b>	
  	
<b>LAGERUNG</b>	
 	

## 10. PACK SIZES / UNITS

8cm x 10cm a` 12 piece

8cm x 10cm a` 60 piece


10cm x 15cm a` 12 piece

10cm x 15cm a` 60 piece


20cm x 30cm a` 6 piece

20cm x 30cm a` 30 piece

  
**AKANNI**  
 Healthcare GmbH  
 Vertrieb durch:  
 AKANNI Healthcare GmbH  
 Krassolzheimer Straße 47, 97346 Iphofen  
 www.akanni.de

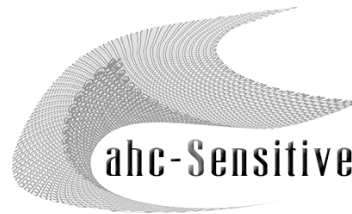


 Avery Dennison Medical Ltd.  
 IDA Business Park, Ballinalee Road  
 Longford N39 DX73, Ireland

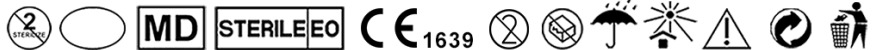

Gebrauchsanweisung lesen, wenn die Verpackung unbeschädigt und ungeöffnet ist.  
 Bitte Gebrauchsanweisung lesen.

## 11. MESSAGE / COMPLAINT

Any serious incident related to the product must be reported to the manufacturer / distributor of the Member State in which the user / patient is located.

Contact: info@akanni.de or +49 9326-9789850

Datum der Überarbeitung des Textes: 2020-07, rev.: 01

**OBSOLETE**